

Electrode cap PROFESSIONAL light

Printed textile cap with a thin wire with preinstalled Ag/AgCl sintered MCScap-NT electrodes with DB-25M common connector.

REF	Size	Head circumference
003-3-175	XL	60-66 cm
003-3-176	XL/L	57-63 cm
003-3-177	L	54-60 cm
003-3-178	L/M	51-57 cm
003-3-179	M	48-54 cm
003-3-180	M/S	45-51 cm
003-3-181	S	42-48 cm



INTENDED USE

The product is designed for positioning electroencephalographic electrodes on the head, recording and transmitting bioelectric potentials to a biopotential amplifier during electroencephalography (EEG).


SET

- Electrode cap PROFESSIONAL light,
- User Manual.

DESCRIPTION


Electrode cap PROFESSIONAL light is made in the lightest version in the line of other electrode caps MCScap®. It uses a textile cap MCScap LIGHT without a chin belt and the thinnest Ag/AgCl sintered electrodes MCScap-NT. The electrode lead wires are assembled into a thin patient cable using a splitter, which is attached to a textile cap. The cap has a DB-25M common connector for connecting to an EEG amplifier. The electrode cap is designed for use with electroencephalographs and biological signal amplifiers.

Electrode cap PROFESSIONAL light can be used for general EEG practice, EEG monitoring, scientific EEG.


 *The electrode cap is not suitable for self-repair.*

Textile cap is made of elastic material, preserving the shape and size. The cap provides the exact position of the electrodes on the head without additional measurements and adjustments. Large holes are provided for ventilation and access to the electrodes and patient's skin. The cap is fixed on the head with the chin or chest belt. The caps are marked according to the 10-10 system. Size identification is carried out by the color of the seam.


MCScap-NT is a cup Ag / AgCl sintered electrode for EEG recording. MCScap-NT is designed for research requiring increased patient comfort for a long time. The Ag / AgCl sintered electrode material guarantees minimum polarization and long-term signal stability, as well as an increased electrode life. The conductive surface of the MCScap electrodes is not in direct contact with the skin. Contact is provided by a conductive substance. A hole in the electrodes is provided to add a conductive gel. Electrode have additional labeling what makes easy to rearrange them to new textile base.

 *It is allowed to change the specification by prior agreement. Please contact the manufacturer for information on available options.*

SPECIFICATION


Types of EEG examinations	general EEG practice, EEG monitoring, scientific EEG
Recommended body position during examination	sitting position, lying position
Electrode	 MCScap-NT
Material of electrode conductive surface	Ag/AgCl sintered
Electrode body material	plastic
Contact substance required	required
Internal diameter of the electrode at the point of contact of the electrode contact substance with the skin	8 ± 0.3 mm
The diameter of the hole in the electrode to add gel	2.8 ± 0.3 mm
Offset voltage	≤ 50 mV
Resistance of electrodes insulation	≥ 1000 MΩ
Dielectric strength of electrodes insulation	1500 V
The impedance of the electrode	≤ 5 kΩ
Number of EEG electrodes / leads	21 / 20
EEG electrode positions	Fp1, Fp2, F3, F4, C3, C4, P3, P4, O1, O2, F7, F8, T3, T4, T5, T6, Cz, Fz, Pz, GND Fp1, Fp2, F3, F4, C3, C4, P3, P4, O1, O2, F7, F8, T7(T3), T8(T4), P7 (T5), P8(T6), Cz, Fz, Pz, GND, Ref
Connector type	DB-25M
Cable length	1.5 m
Location of the output of the electrode cable from the cap	back of the head
Ear electrodes	no
Marking of the textile cap	yes
Marking of the electrodes	yes
Weight of EEG cap	< 250 g
Net weight	< 750 g
Gross weight	< 800 g

PIN LAYOUTS OF COMMON CONNECTOR

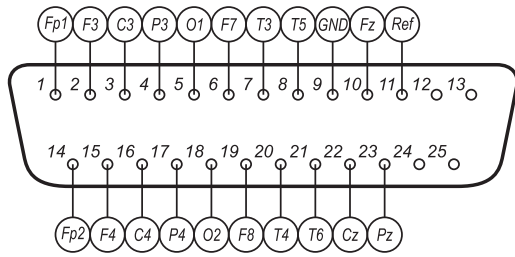

#23-01 (21 electrodes)

SCHEME OF ELECTRODE ARRANGEMENT

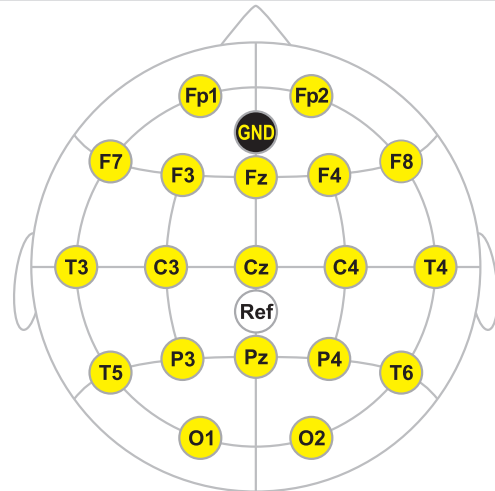

[[[Undefined variable Global(External)Variables.term Код схемы
производителя]]]: 23M21 (21 electrodes)

For information about other possible EEG electrode arrays, contact the manufacturer.