# EEG accessories MCScap

## MANUAL FOR CLEANING AND MAINTENANCE TEXTILE CAPS AND ELECTRODES MCSCAP

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MCS.MC000000-63EU-1. #2021-01-12

EEG accessories MCScap  
Manual for cleaning and maintenance textile caps and electrodes

EN

Textile caps and electrodes MCScap are designed for non-invasive registration EEG.

Full User Manual is available on [mcs.scap.com/support](https://mcs.scap.com/support)

### CLEANING

Immediately after each test clean cap and electrodes using water and mild detergent. If necessary, use a brush from the kit or a soft toothbrush. Rinse with clean water and dry on a towel or hang.

 **Avoid to put the connector in fluid!**

See video *Cleaning of EEG accessories* on [www.mcs.scap.com/video](https://www.mcs.scap.com/video).

### DISINFECTION AND STERILIZATION

For disinfection products it is recommended to use 70% solution ethanol or use disinfectant solution Gigasept FF NEU or Bomix Plus. Take notice of instruction for use the disinfection solution.

Gas sterilization with ethylene oxide is acceptable.

 Do not use disinfectants with means of mass content shares of isopropyl alcohol more than 5%.

 Do not use autoclave or other hot sterilization methods.

### STORAGE

Temperature: from +5°C to +40°C. Humidity: from 20% to 80%.

 Don't expose direct sunlight.

### UTILIZATION

This device must be disposed according to regulations of local legislation.

### SYMBOLS ON LABELS

 Wash by hand	 Washing, mild process, maximum 30 Celsius
 Do not tumble dry	 Do not wring

### WARRANTY

Producer guarantees this product is realised in compliance with 93/42/EEC directive. Producer declines any responsibility, is associated with improper use of the product or when it used differently to its intended purpose.

The warranty period is six months. The warranty period is calculated from the date of purchase of the product.

The lifetime of the product is 18 month when following the recommendations and instructions contained in the User Manual. The lifetime of the product is calculated from the date of its purchase, or if it is impossible to determine this, from the date of manufacture of the product.

The storage period of the product should not exceed 24 month before it starts to use.

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